

## **INSTRUCTIONS FOR USE DIASHINE POLISHING COMPOUND (EXTRA ORAL)**

### **PURPOSE:**

DiaShine Polishing Compound (Extra Oral) is a water soluble diamond polishing compound available in various grits used to create a variety of surface finishes from matte to high lustre in ceramics, metals and composites.

### **DIRECTIONS:**

Choose a preferred rotary applicator tool (bristle brush, felt, cotton wheel) and run it through the paste in the jar at a speed of around 5k rpm - the friction will melt the paste onto the applicator tool. Do not remove paste from the tub or add water - there is no need.

Once the applicator tool has a layer of paste on the outside (edges) and starting at a slow speed (around 5k), spread the paste onto the desired surface and begin polishing keeping the tool in contact with the surface. Speed up as needed. Within a few seconds the finish will begin to appear. There are several methods to polishing - side to side, front to back, pull stroke, but keep the applicator in contact with the surface.

DiaShine polishes are all completely water soluble for easy cleanup.

### **CONTRAINDICATIONS:**

Used in the proper way as stated above, minimal waste is achieved and no adverse effects should be realised.

### **PACKAGING:**

Black Jar

### **STORAGE:**

Store in a dry place preferably at room temperature. Keep away from extreme heat. If left in direct sunlight for a long period of time, paste may become slightly viscous and sticky. Left at room temperature, the material will revert to normal form.

Always replace cap after use.

Keep out of reach of children.

For use by dental professionals.